

NuMED Bare And Covered CP Stent Mounting Procedure

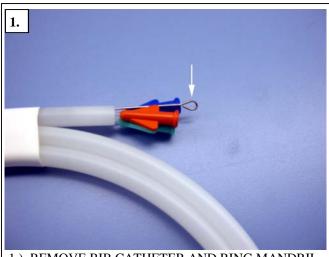

1.) REMOVE BIB CATHETER AND RING MANDRIL FROM PACKAGING HOOP




REMOVE CP STENT FROM THE PACKAGING VIAL

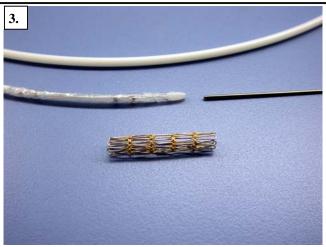


3.) REMOVE BALLOON PROTECTOR FROM BIB CATHETER AND ASSEMBLE MATERIALS (BIB CATH, RING MANDRIL, & CP STENT)



4.) INSERT THE RING MANDRIL THROUGH THE CP STENT AND INTO THE GUIDEWIRE LUMEN OF THE BIB CATHETER.

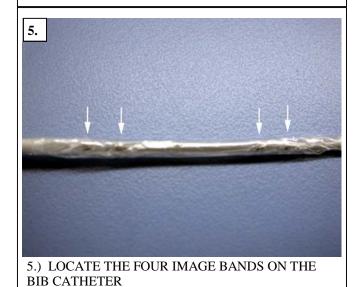

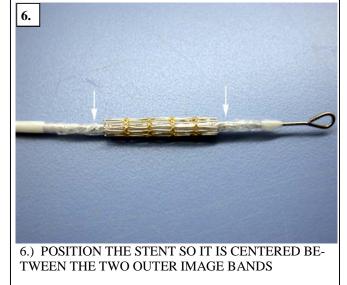




7.) CRIMP THE STENT BY COMPRESSING IT TO THE BALLOON WITH PRESSURE GENERATED WITH THUMB AND FOREFINGER.




9.) INSURE THAT THE CENTRAL AREA OF THE STENT IS CRIMPED.

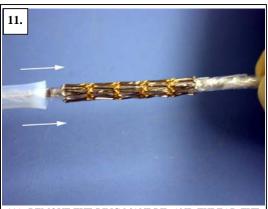


11.) REMOVE THE RING MANDRIL AND THREAD THE SIZING TOOL OVER THE MANDRIL. REPLACE THE MANDRIL INTO THE INNER LUMEN. THE FLARED END OF THE TOOL SHOULD BE FACING THE STENT.




8.) ROTATE THE STENT 90 DEGREES AND REPEAT CRIMPING MOTION. REPEAT THIS STEP UNTIL ENTIRE CIRCUMFERENCE OF THE STENT IS CRIMPED.

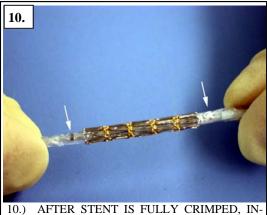

10.) AFTER STENT IS FULLY CRIMPED, INSURE THAT STENT IS STILL CENTERED BETWEEN THE TWO OUTER IMAGE BANDS.

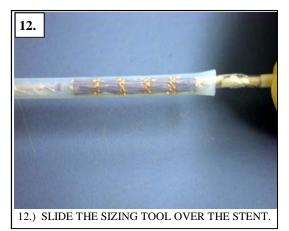

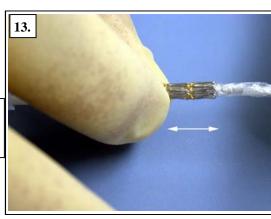

13.) REMOVE THE SIZING TOOL AND GENTLY INSURE THAT THE STENT IS TIGHT ON THE BIB CATHETER.